Actreen® Mini Cath

EN Instructions for use
DE Gebrauchsanweisung
ES Instrucciones de uso
FR Mode d'emploi
IT Istruzioni per l'uso
NL Gebruiksaanwijzing
AR تعليمات الاستخدام
BG Инструкции за употреба


CA Mode d'emploi
CS Návod k použití
DA Brugsanvisning
EL Οδηγίες χρήσης
FI Käyttöohje
HU Használati útmutató
LV Lietošanas instrukcija
MS Arahan penggunaan

NO Bruksanvisning
PT Instruções de Uso
PL Instrukcja użycia
RO Instrucțiuni de folosire
RU Инструкция по применению
SK Návod na použitie
SV Bruksanvisning
TR Kullanma talimatları

EN Instructions for use

Actreen® Mini Intermittent Urinary Catheters are indicated for intermittent urinary catheterisation by female patients with chronic urine retention or voiding dysfunction.


Actreen® Mini Cath is a sterile, pre-lubricated, ready-to-use catheter.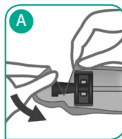

For safety and hygienic reasons, we advise you to use the product in shortest possible time after opening.


Preparation:

- 1- Before opening, check the product is intact and ensure the lubricant is evenly spread over the catheter (visual and tactile inspection).
- 2- Wash your hands very carefully before and after catheterisation, and wash the genitourinary area.
- 3- Remove the protective covering on the connector side of the catheter by tearing at mark $\blacktriangleleft \blacktriangleright$. **A**
- 4- Hold the catheter by the connector to remove it from the protective covering without touching the catheter. **B**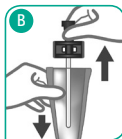

Catheterisation:

- 5- With one hand, separate the labia and with the other, delicately insert the catheter through the urethral opening into the urethra and up to the bladder, until urine starts to flow. **C**
- 6- Manual pressure on the lower abdomen towards the end of catheterisation will ensure that the bladder has been completely emptied.
- 7- When the bladder is completely empty, gently withdraw the catheter.
- 8- The catheter may then be discreetly replaced in the protective covering and be discarded according to the current local regulations **D**

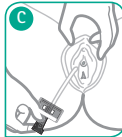Actreen® Mini Cath can be attached to a urine collection bag.

Self-catheterisation should only be carried out under medical advice and only in accordance with the instructions given.

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B. Braun Medical AB
B. Braun Medical Inc.
B. BRAUN MEDICAL SAS | 26 Rue Armengaud | 92210 Saint-Cloud | FRANCE